



This product is ideal for patients with arthritis and limited bending motion. Donning devices are designed to assist the patient in correctly applying JINNI® compression stockings. Use with closed as well as open toe compression stockings.

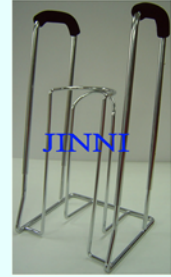

Allows for a simple step-in procedure shown below for donning stockings.

1. Place your JINNI® compression stocking inside the semi-circle frame and pull the top of the stocking down over the semi-circle. The heel of the stocking must face the back of your JINNI® Stocking Donner.
2. Continue to slide the stocking down over the semi-circle until the heel is centred and even with the top of the semi-circle. (It may be necessary to pull the heel farther down depending on foot and stocking size.)
3. While sitting, insert your foot into the stocking until your foot is completely on the floor. Hold the heel pocket against the frame as you slide your toes into the stocking.
4. Grasp the padded handles and gently pull the JINNI® Stocking Donner and stocking up the leg until the stocking is above the calf. Try not to over-stretch the stocking.
5. Pull the JINNI® Stocking Donner backward and down, free of the stocking.
6. Be sure the heel of the stocking is properly placed on the foot, then adjust the length and smooth out any wrinkles by stroking and smoothing with the palm of your hands.

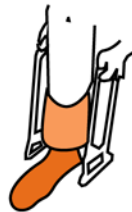

Thigh and waist high stockings can also be donned using your JINNI® Stocking Donner with the lower leg applied in the same manner as a knee length and then pull the remaining portion into place by hand.